

PALACOS® R+G pro

Quick-Guideline

PREPARATION:

1. Prepare the sterile PALAMIX® cement gun for use.
2. Connect the PALAMIX® vacuum pump and vacuum hose.
3. Do not remove the PALACOS® R+G pro pack from the aluminium packaging until immediately prior to use.


HANDLING:



Loosen the cement powder in the cartridge: Move the mixing rod up and down twice.



Press the sealing ring down into the cartridge and ensure that it is pressed in fully so that the cartridge is closed vacuum-tight. Then connect the vacuum hose to the sealing ring.

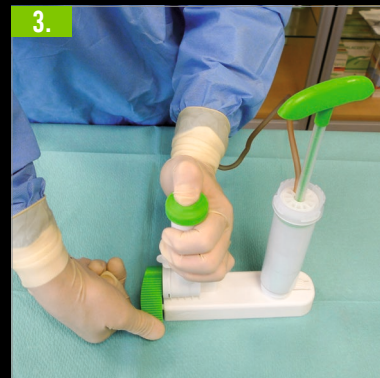


Press on the green vacuum indicator and turn the green scoring lever quickly forwards as far as it will go. Caution: The green scoring lever must click twice (2x click); carrying out this procedure in the tray is prohibited.

- 4.**
- IMPORTANT:**
- A. Operate the vacuum pump; monomer transfer concludes after 10 seconds.
 - B. Continue to operate the vacuum pump. Mix the bone cement for at least 20 seconds until a homogeneous dough develops.
 - C. Remove the vacuum hose from the sealing ring.




Pull the mixing rod upwards and break it off at the mixing head at the predetermined breaking point.

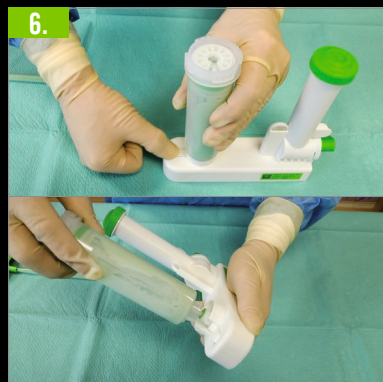

Press the cartridge attachment mechanism downwards and unscrew the cartridge.



Screw the application nozzle onto the cartridge.

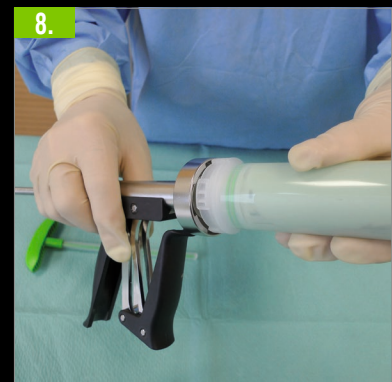

Place the cartridge into the PALAMIX® cement gun.